



Proper Output Jack Connection

It's important to correctly insert cables into the output jacks to avoid damage.

1. Avoid using excessive force to insert plugs into the jacks. High levels of force can cause the jacks to become loose and compromise functionality. Gently insert the cable tip straight into the output jack.
2. If possible, mount the noddle in a space where it cannot be bumped into. Outside forces from an accidental bump can bend the tip of the cable and possibly damage the output jack.
3. After plugging a cable into an output jack, be sure that noddle is mounted in such a way that if the cable is pulled, the cable will pull straight out and not torque the jack.
4. When unplugging a cable from an output jack, gently pull straight out by grabbing close to the tip of the cable. Do not jerk the cable out of the jack.
5. Avoid having the noddle mini USB charging cord torqued to the side. Do not jerk the cable out of the USB port. Gently pull straight out by grabbing the base of the cable.

